

## AED INSPECTION CHECKLIST - SAMPLE

AED Inspection			
AED Nickname test1	AED Model Zoll   AED Plus	Serial Number G00G000000	Site testsite1
<b>A. AED</b>			
<input type="checkbox"/>	AED Readiness indicator displaying green or "OK".		<input type="text"/>
<input type="checkbox"/>	AED pads or cartridge attached in unopened pouch and free of damage or foreign debris.		<input type="text"/>
<input type="checkbox"/>	Battery Pack <u>installed</u> in AED		<input type="text"/>
<input type="checkbox"/>	No "Errors" or "Alarms" noted.		<input type="text"/>
<input type="checkbox"/>	Spare Pads in AED carrying case pouch or under cover. <input type="checkbox"/> Click here if not equipped.		<input type="text"/>
<input type="checkbox"/>	Pediatric Pads/Key/Switch Present. <input type="checkbox"/> Click here if not equipped.		<input type="text"/>
<b>B. RESPONDER KIT INCLUDES</b>			
Kit should include: Pocket mask or shield (1), Disposable Gloves (2 pair), Razor (1), Scissors (1), Towel (1)			
<input type="checkbox"/>	Kit Complete and attached to the AED		<input type="text"/>
<b>C. ALARM BOX</b>			
<input type="checkbox"/>	Check Cabinet alarm and replace battery every 12 months. <input type="checkbox"/> Click here if not equipped.		<input type="text"/>
		<input type="button" value="Submit"/>	<input type="button" value="Cancel"/>